

ProClude[®]

Desensitizing Prophylaxis Paste

With SensiStat[®]

Clinically proven to reduce sensitivity after SRP treatment

CAUTION: US Federal Laws restrict this device to sale by or on the order of a dentist.

These instructions cover the following product:
1 - 4 oz. tube; mint. Tube contains enough product for approximately 60 procedures. ProClude Prophylaxis Paste is available in the following grit:

Medium

Suitable for most cleaning procedures where a high level of polish is desired.

INDICATIONS FOR USE

To be used for cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. Secondly, ProClude can be used for the management of tooth sensitivity, post-scaling and root planing. For staining not responsive to removal by ProClude, use an appropriate prophypaste with an appropriate abrasive characteristic to remove the stain followed by the use of ProClude.

COMPOSITION

ProClude Prophylaxis Paste contains hydrated silica, calcium carbonate, glycerin, arginine bicarbonate, water, flavor, cellulose gum, sodium saccharin, FD&C blue no. 1. It is a unique blend of polishing, cleaning and dentinal tubule sealing agents designed for professional application during standard dental practice hygiene procedures.

CONTRAINDICATIONS

ProClude Prophylaxis Paste is contraindicated in patients with a known allergy to any of the components.

WARNINGS

ProClude Prophylaxis Paste may cause eye irritation upon contact. If contact with eyes occurs, immediately flush eyes with copious quantities of water for 15 minutes.

Seek medical attention if irritation or discomfort persists.

PRECAUTIONS

ProClude Prophylaxis Paste should be used only by individuals professionally trained to perform dental prophylaxis. During use, wear protective glasses and gloves.

STEP-BY-STEP INSTRUCTIONS

Sufficient paste for one procedure should be removed and placed in a clean dampen dish or other suitable container.

Using a disposable prophy angle on a conventional, metal prophy angle, fill the cup by pressing face of the cup into the paste while running the angle at low speed.

Apply the paste to the teeth – **especially at the cervical surfaces where most tooth sensitivity occurs. Rinse after application.**

NOTE: Best results are obtained when paste is burnished into cervical sites where teeth are or can become sensitive.

Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

STORAGE

Close cap immediately after use to avoid moisture loss. Store at room temperature. Do not store pastes above 25°C. Keep out of sunlight.

LOT NUMBER AND EXPIRATION DATE

The Lot Number should be quoted in all correspondence which requires identification of the product. Do not use after expiration date.

Colgate Oral Pharmaceuticals, Inc.

*a subsidiary of Colgate-Palmolive Company
New York, NY 10022, USA*

*Questions? Comments?
Call toll-free 1-800-962-2345*

Rev. October 2007

P10000410